



TEST PROCEDURE


How to perform the test

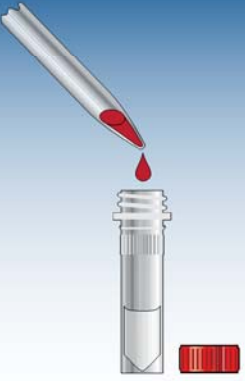


1. Mix blood collection tube by inverting 2-3 times. Add 50 μ L of sample to *illumigene®* Buffer I. Mix by inversion 5 times and hold for 2 minutes.




2. Mix prepared sample by inversion 5 times and immediately transfer 50 μ L to SMP PREP IV. Mix the SMP PREP IV by inversion 5 times.



3. Remove the tip cap of SMP PREP IV and gently squeeze to collect 5-10 drops into a clean Tube I.

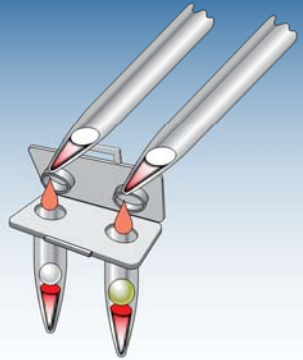


4. Using a new pipette tip for each, transfer 50 μ L of the prepared sample to both the TEST and CONTROL chambers of the *illumigene®* Malaria Test Device.

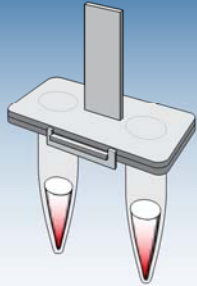

5. Close and fasten the latch securely. Gently tap device to remove air bubbles trapped on the bottom of the tubes. Carefully examine the reaction tubes to ensure that there are no air bubbles left in the tube.



6. Insert *illumigene®* Test Devices into *illumipro-10™* and initiate amplification reaction and detection by selecting the Malaria program.

 This illustration is representative of the current Package Insert at the time of publication. Please refer to the most current version of the Package Insert for complete instructions.

© 2015 SN11113 REV. 12/15

USA/CORPORATE OFFICE
3471 River Hills Drive
Cincinnati, Ohio 45244 USA
Telephone: 513.271.3700

Orders / Customer Service: 1.800.543.1980
Technical Support: 1.800.343.3858
Information Fax: 513.272.5432
Ordering Fax: 513.271.0124
www.meridianbioscience.com

MERIDIAN BIOSCIENCE EUROPE
Belgium/Luxemburg Tel: +32 (0) 67 89 59 59
France Tel: +33 (0) 1 42 56 04 40
Germany Tel: +49 (0) 3371 681 500
Italy Tel: +39 0331 43 3636
Netherlands Tel: +31 (0) 411 62 11 66
United Kingdom Tel: +44 (0) 20 8453 7970

 **Meridian**
Bioscience, Inc.
Inspired Science. Trusted Solutions.™